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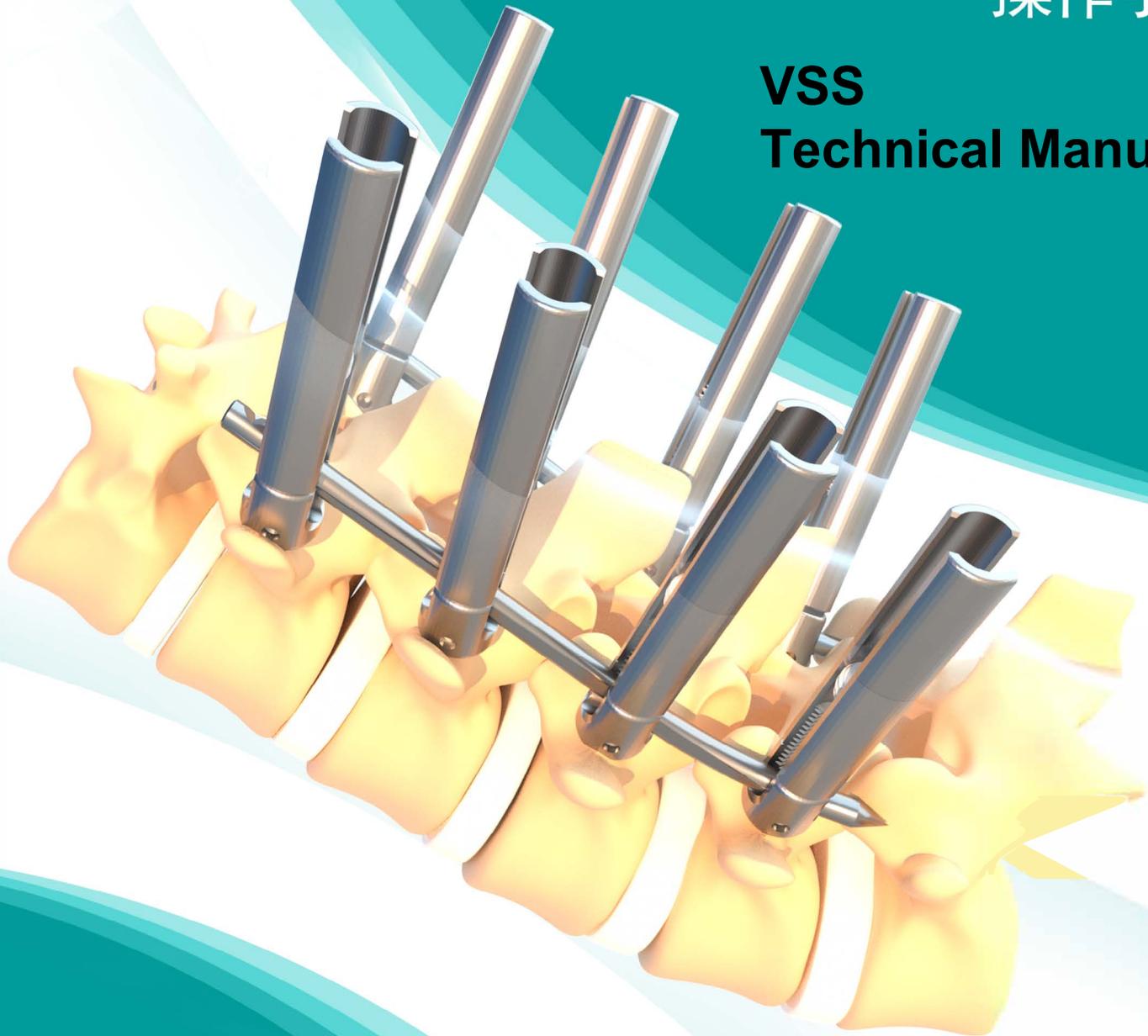
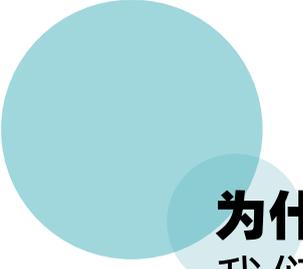
Beijing Fule is a national high-tech enterprise integrating R&D, production and sales of medical equipment, and has a production line of fully intelligent processing equipment.

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The academician expert studio was established to help Fule improve its research and development capabilities, and further deepen the cooperation of production-academy-research; it was approved as a post-doctoral research station.

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The hardware facilities are complete, the R&D team is excellent, and the clinical experts work closely with more than one hundred domestic and foreign patents.

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Based on the agent cooperation model, it has established a nationwide sales and service network, and its products are supplied to nearly a thousand tertiary hospitals across the country and exported to more than 20 overseas countries.

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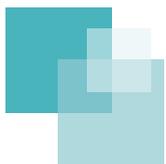
Long tail design, large visual space, small incision, and less bleeding

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Double thread design, adapt to vertebral bone density distribution, stronger holding power

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Cannulated bone cement hole, can be used for osteoporosis

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Easy to distraction and compression

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Dedicated rod holder, convenient and accurate

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Include straight rod, 50-260 mm, Bending rod, 50-200 mm

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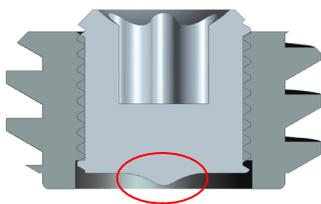
Double screw nut design to avoid loss of reset

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Step:
1. tighten outer nut for screw
2. tighten inner nut for rod

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Note:
Before tightening the double nut, please make sure that the bottom of inner nut must not exceed the outer nut.

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Elderly patients with severe cardiopulmonary disease
Patients with severe osteoporosis
Patients with significant scoliosis
Patients with pedicle dysplasia
Patients with spondylolisthesis above degree II

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Surgical Technical

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Step 1. Position preparation, body surface positioning

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This operation needs to be done with the help of X-ray fluoroscopy, so the patient needs to be prone on a fluoroscopy operating table and maintain full fluoroscopy in the front and side positions (1a)

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After the patient is anesthetized, the fractured vertebral body can be partially or completely reduced by adjusting the position

Determine the body surface position of the punctured pedicle (1b)

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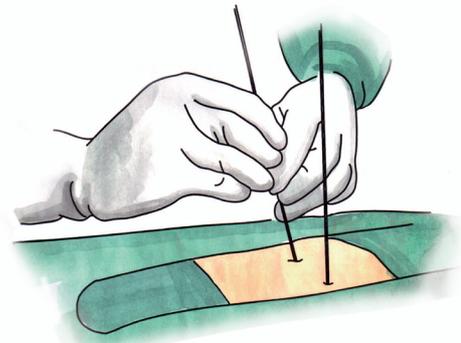
Pre-puncture to determine the incision location, the location of the puncture point: 1-1.5cm outside the pedicle body surface position, and the approach of obese patients needs to be deviated to the outside (2a)

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Local infiltration of epinephrine saline at the incision site (3a)

incise the skin and fascia with 1.5 cm incision (3b)

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Insert the positioning guide puncture needle, the insertion process needs to confirm the position and direction of the positioning guide puncture insertion through fluoroscopy (4a).

Under the guidance of anteroposterior X-ray fluoroscopy, place the tip of the positioning guide trocar on the outer edge of the pedicle projection. (I.e. 9-10 points on the left side and 2-3 points on the right side), puncture into the vertebral body with an inwardly inclined 10-15° parallel to the endplate, positioning and guiding the puncture into the bone 6-10mm and fluoroscopic positioning and guiding The tip of the trocar did not break through the medial cortex in the pedicle projection (4b).

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Insert the pin into the puncture needle. The pin should exceed the puncture needle by 35-40 mm and bone fixation has been obtained (5a). The pin should be deeper than the cannulated tap.

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When the pin reaches the desired depth, the puncture needle can be removed (5 b).

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First put the dilator (small) within pin and insert it into the bone surface, and then insert the larger dilator in sequence. When all the dilators are ready, remove the dilator (small) (6a).

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Along with the pin, use cannulated drill to bore the cortical bone(6b)

Select the appropriate cannulated tap. Pay attention to the depth when tapping, and note the tapping depth (6c)

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Take out the second dilator, select the pedicle screw of appropriate length and diameter, and implant it in through the pin. Pay attention to confirm the position of the pedicle screw under fluoroscopy monitoring.

After pedicle screw is implanted, remove the pedicle screwdriver, and take out the pin.

When the upper end of the pedicle screw is flush with the upper end of the dilator, it proves that the pedicle screw is implanted correctly (7a).

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Select a suitable rod, insert the tail of the rod into the small hole of the rod holder, and make the platform of the tail of the rod face up (8a).

Insert and rotate the locker of rod holder until the rod is tightened (8b).

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Surgical Technical

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Step 9. rod placement

The rod should be entirely inside the pedicle screw long tail. Slowly lower the rod along the long tail until it touches the end of the screw or until the deepest position that the surrounding tissue can allow it to reach. At least the head of the rod needs to be under the fascia (9a).

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Rotate the rod holder with 90 degrees. This step will guide the rod into the bottom of the long tail. To confirm that the rod is located in the bottom of the long tail, intraoperative fluoroscopy can be used to confirm the placement of the rod (9b).

If the arc of the rod does not meet the expected requirements, we can use rod bender to ensure the proper arc of the rod (9c).

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After confirming that the rod is located in the long tail, use nut holder to insert the nut and tighten the nut with distal pedicle screw (10a).

In order to prevent rotation and height loss of the vertebral body during expansion and compression, all distal screw nut need to be tightened (10b).

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【步骤十一】 撑开、加压并最终锁紧 Step 11. distraction, compression, and final tightening

The distractor can realize distraction and compression at the same time. A universal angle can be realized between the two rotating sleeves, which can be well adapted to the expansion and compression of angled pedicle screws.

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Install extend level on the pedicle screw, and then adjust the angle of extend level according to the angle of pedicle screw to be distracted or compressed (11a).

Use 2 distractors and wrench for distractor to make distraction and compression, and then tighten all nuts in best reset state(11b).

Use 10 N.m Handle and Counter Torque to make sure all nuts are fully tightened (11c).

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Remove all long tails with Break-off wrench(12a).

After all tails are taken out, suture the wound.

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Products information

Polyaxial Reduction Cannulated Pedicle Screw

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Diameter

Polyaxial Reduction Cannulated Pedicle Screw (Uniplanar)

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Diameter

Monoaxial Reduction Cannulated Pedicle Screw

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Diameter

Polyaxial Reduction Cannulated Pedicle Screw (Double Nut)

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