



**LANNX Med & Bio Co., Ltd.**, office located in Shenzhen city (The high-tech center of China). LANNX is a leading oxygen supply manufacturer who focuses on researching, manufacturing and distribution of hyperbaric oxygen chamber, oxygen concentrator and other oxygen related products.

LANNX's purpose is to provide the clients with professional products and services, improve the customers' experience; promote high-standard life and health quality, delivery and share health with the world.

We can be your best partner in healthcare area, end to end solution will help you improve your business growth and profit improvement.

**Trademark : “DR.HUGO “and “LANNX”**

The logo for DR.HUGO, featuring the word "DR.HUGO" in a bold, blue, sans-serif font. The letter "D" is partially yellow, and the letter "O" is blue with a yellow molecular structure icon inside it.

**For soft TPU chamber that you need to know before inquiry:**

**Recommend Single soft Chamber:**



uDR L1



uDR L2



uDR K1




uDR S2




1. Normally the price we quoted based on one unit of the chamber, the chamber body with oxygen concentrator. We will match with all the necessary accessories like the oxygen tubes/values/connectors to make sure the chamber works well.

2. If you want to buy **separate chamber body or concentrator**, we can accept this kind of order.
3. We can do **different pressure version for soft chamber** and the price would be different. But most of the clients would choose 1.3ATA. If you don't require pressure, we will just quote the price for 1.3ATA.
4. For the concentrator, usually we will match with O3 concentrator(10L) for most of the chambers. **O3 concentrator match with remote controller**, the inside person can control the concentrator by remote controller, the outside person can control the concentrator by the machine panel.

Model No.	Key Features	Package	MOQ	Picture
uMR O3	<p>Features:</p> <ol style="list-style-type: none"> <li>1. Oxygen concentrator and Compressor combine in 1</li> <li>2. The oxygen generator uses air as the raw material and generates a certain concentration of oxygen (98%±3%) through the principle of molecular sieve pressure swing adsorption (PSA method).</li> <li>3. The compressor is Oil-free compressor air delivery system</li> <li>4. The oxygen system can provide chamber a maximum pressure of 50Kp (gauge pressure) or 1.5ATA, as well as high purity oxygen;</li> <li>5. The oxygen system not only provides oxygen to the space, but also provides clean air.</li> <li>6. Timing control: With a display screen, the timing function is easy to use and easy to operate.</li> <li>7. Overheating protection: The pure oil-free compressor is equipped with an overheating protector, which will automatically stop working when the operating temperature of the compressor is too high, so as to better protect the compressor and the safety of the whole machine.</li> </ol> <p>Specification:</p> <p>Model: uMR O3            Size: 640mm*630mm*355mm            Control System: Button + LED Display            Power Supply: AC 100V-240V 50/60Hz            Power: 1200W            Oxygen Pipe Diameter: 8 mm            Air Pipe Diameter: 12 mm            Oxygen flow: 10L/min            Max. air flow: 205 L/min</p>	<p>1PCS/ Carton            Carton Size: 54*83*94cm            Carton Weight: 75KG</p>	1	

If you want to update the concentrator to the other concentrator model, we accept it. We suggest that you can choose **O7 concentrator(10L) which match with inside chamber control panel**. The inside user no need the remote controller, they can handle the panel directly. We will match with all the wire cables and connector, please don't worry.

<p>2 in 1 hyperbaric oxygen supply system (10L oxygen concentrator-compressor) 10L 单压呼吸机 适用于单人一体箱体 (C3 MINI) 氧舱供氧。默认带舱内控制屏</p>	<p>uMR O7</p> <p>Features: 1.Oxygen concentrator and Compressor combine in 1 2.Touch screen UI interface for index setting and display. 3.Support to connect with remote control panel. 4. Intelligent control, the pressure in the cabin can be adjusted on the UI interface 5. The oxygen generator uses air as the raw material and generates a certain concentration of oxygen (96%±3%) through the principle of molecular sieve pressure swing adsorption (PSA method). 6. The compressor is Oil-free compressor air delivery system 7. The oxygen system can provide chamber a maximum pressure of 50Kp (gauge pressure) or 1.5ATA, as well as high purity oxygen. 8. The oxygen system not only provides oxygen to the space, but also provides clean air. 9. Timing control: With a display screen, the timing function is easy to use and easy to operate. 10. Overheating protection: The pure oil-free compressor is equipped with an overheating protector, which will automatically stop working when the operating temperature of the compressor is too high, so as to better protect the compressor and the safety of the whole machine. 11. SOS alarm: Equipped with SOS one-key alarm function, which can provide danger warning and ensure safe use.</p> <p>Specification: Model: uMR O7 Size: H767.7*L420*W400mm Control System: Touch screen control (7 inch)</p>	<p>1PCS/ Carton Carton Size: 46*49*98CM Carton Weight: 58KG</p>	<p>1</p>	
--	---	---	----------	---



5. According to the clients' orders, we choose the popular size and shape and named it in our model number. You can ask for catalog to compare the sizes. Usually we have stock goods for our model. If you want to **custom the size or shape**, we can do it and the lead time is about 7-10 days.

**6. We accept custom logo on the chamber** and it would have extra cost. If you don't have special requirements, what we ship would be our stock chamber which print with our own logo.

## The optional equipment of soft chamber:

### 1. Inside control panel.

As what we mentioned, O7 concentrator have the inside control panel. If you update the concentrator into O7, you can get the inside control panel. We will match with the hanger and wire cables.



### 2. Air cooler.

The inside chamber temperature will be higher 1-2 °C than room temperature during oxygen therapy. For some clients who can't bear inside chamber warm temperature, air cooler is a good choice for them. We will match with all the connecting tubes of the air cooler.

## Mini Air cooler

Small size, environmental protection and pollution-free intelligent control heat exchanger temperature display and adjustable easy installation

**Model:McQ A3**



This product is a air condenser for electronic refrigeration. It is used to cool and dry the hot and humid air to reduce the water content of the sample air. It adopts freeze drying and is mainly used to dry and purify the air source at the air terminal to achieve the purpose of air-liquid separation. This device is mainly used as a pretreatment device for various gases.

It can reduce the dew point of the air and reduce the problem of water in the air to the used devices and equipment. This product adopts precise temperature control, and the temperature is adjusted through the intelligent module to keep the temperature of the cold chamber constant at the set temperature, and the temperature accuracy is 0.1 C Drainage device cycle timing work control time 0-9999 seconds.



### 3. Micro air conditioner.

If you are not satisfy the air cooler effect, we suggest that you can choose micro air conditioner. The inner AC is small which won't occupy with much space. And it has better cooling effect for inside chamber.



#### 4. Bracket/Frame.

We have the bracket/frame for the chamber, but the price we quoted is not including the bracket normally. Because the size is big, the shipping cost would be higher. We suggest that you can custom the bracket/frame in local hardware store. But if you can accept the shipping cost, we can prepare the bracket/frame for you.

<p>Simple outer braket- disassembled and assembled Material: stainless steel + paint</p>	<p>71KG</p>	<p>1</p>	
<p>complicated outer braket- disassembled and assembled Material: stainless steel + paint</p>	<p>95KG</p>	<p>1</p>	

**Please noted that all the optional equipment would have extra cost. Please contact our sales for the accurate cost.**

If you want to see more details, you can visit our Youtube channel:

<https://www.youtube.com/@oxygensupplycenter>


The playlist about our soft chambers:

[https://www.youtube.com/playlist?list=PLj8Yy8cT\\_JqVjpaJLvn7iimEEa-gZ6DdB](https://www.youtube.com/playlist?list=PLj8Yy8cT_JqVjpaJLvn7iimEEa-gZ6DdB)





**For Hard Steel Cabin chamber that you need to know before inquiry:**

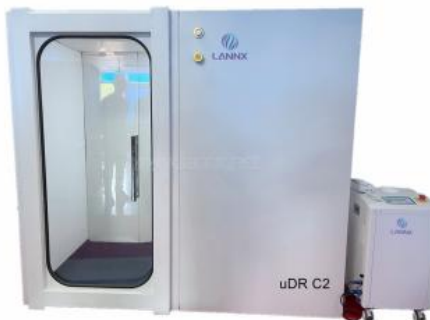

1. All the cabin chamber have the same appearance, just different sizes. C3 series have stock goods. If you want C2 series, you need to wait one more week. The following are all the chamber sizes we have:

**uDR C2 Mini:** 135(L)\*78(W)\*191(H)cm ➔ Door size: 137(H)\*47(W)cm

**uDR C2:** 200(L)\*100(W)\*180(H)cm ➔ Door size: 150(H)\*65(W)cm

**uDR C2 Max:** 200(L)\*170(W)\*180(H)cm ➔ Door size: 150(H)\*65(W)cm

**uDR C2 Huge:** 200(L)\*280(W)\*180(H)cm ➔ Door size: 150(H)\*65(W)cm

**uDR C3 Mini:** 135(L)\*78(W)\*180(H)cm ➔ Door size: 137(H)\*47(W)cm

**uDR C3 STD:** 175(L)\*88(W)\*188(H)cm ➔ Door size: 149(H)\*55(W)cm

**uDR C3:** 180(L)\*100(W)\*190(H)cm ➔ Door size: 150(H)\*66(W)cm


**uDR C3W:** 220(L)\*120(W)\*190(H)cm ➔ Door size: 149(H)\*66(W)cm

**uDR C3W+:** 140(L)\*80(W)\*188(H)cm ➔ Door size: 140(H)\*51(W)cm

**uDR C3 Max:** 220(L)\*180(W)\*190(H)cm ➔ Door size: 149(H)\*66(W)cm

**uDR C3 Huge:** 220(L)\*240(W)\*190(H)cm ➔ Door size: 149(H)\*66(W)cm

**uDR C4:** 220(L)\*300(W)\*190(H)cm ➔ Door size: 149(H)\*66(W)cm

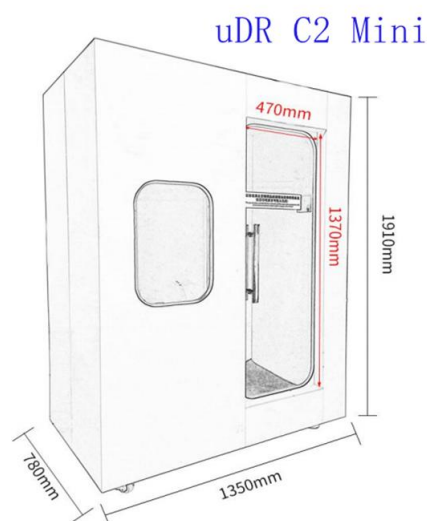



2. Usually the price we quoted based on one unit chamber(chamber body + oxygen concentrator + sofa + humidification bottle + intercom + light strip + other accessories) that works well. We will match with all the necessary accessories to ensure the chamber can be workable.





3. For the oxygen concentrator of cabin chamber, we match with **O7/O7+ concentrator(10L) which has touch screen panel**. And we will match with inside control panel, the user can control the concentrator during oxygen therapy. If you want to update the concentrator, we also have higher capacity concentrator like 15L/20L/30L.

4. Because the cabin chamber is large, the packing size is big and we use wooden cartons to pack the chamber. Some airline is not allow the big size carton box and the air shipping cost would be expensive. Therefore, **most of the clients would choose sea shipping channel for cabin chambers**.

5. For cabin chamber, it has comfortable sofa and the customer experience would be great. But you need to assemble the cabin chamber except for uDR C2 Mini/uDR C3 Mini. We have the videos to teach you how to assemble it step by step:

### **Playlist about how to assemble the cabin chamber:**

[https://www.youtube.com/playlist?list=PLj8Yy8cT\\_JqWwgS5r6-hljP3kGobic2kR](https://www.youtube.com/playlist?list=PLj8Yy8cT_JqWwgS5r6-hljP3kGobic2kR)

1). Chamber body installation: <https://youtu.be/DwiU8TjCWDE>

2). Safety valve and connector: <https://youtu.be/maJzRODVq2c>

3). Chamber test: <https://youtu.be/hvOiBAYP-54>

4). Decoration board/light bar/humidification bottle intallation:

<https://youtu.be/6E4MqMRryn8>

5). Air conditioner installation: <https://youtu.be/WAhyibZDteo>

6). Massage chair installation: <https://youtu.be/NarKuSQfogg>

7). DC TV installation: <https://youtu.be/zTGBbCbYONs>

For mini chamber: [https://youtu.be/KL-k\\_kBiUoQ](https://youtu.be/KL-k_kBiUoQ)

## The optional equipment of hard chamber:

If you want to improve the customer's experience, you can consider the optional equipment as below:


### 1. DC water-cooled air conditioner.

Don't use AC appliance product inside the chamber, or it would cause the fire. Our DC air conditioner is similar to household air conditioner, but it is not the same type air conditioner. Household air conditioners are refrigerated with fluorine. As we known, the chamber body is airtight space, it will lead to fluorosis if the fluorine of AC leak out. Our air conditioner is water-cooled conditioner which was designed for the hyperbaric chamber. It was refrigerated by water vapor, does not use CFC-containing compressors. It is no risk of toxic gas leakage, safe, environmentally friendly and energy-saving.





**Water-cooled air conditioner have external unit and internal unit.** Maintain the air conditioner is easy. For external unit, you just need to pour the water one time, no need to pour the water in future. Usually there have the control panel for outside using. Inside user control the air conditioner by remote controller. If you want to install the control panel inside the chamber, it is ok for us.

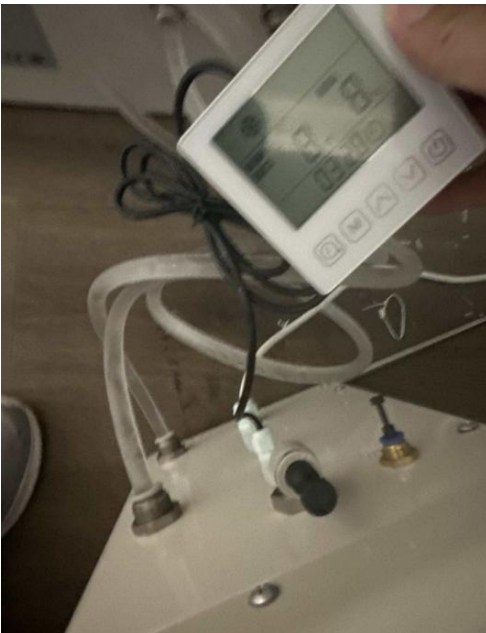

This is the video about how to install the air conditioner:

<https://youtu.be/WAhyibZDteo>

## 2. DC television.

Our TV is DC appliance product, too. So it can be used inside the hyperbaric chamber. You need to buy the local set top box to connect local signal to watch TV. Certainly, you can download the APP to watch videos by internet.

This is the video about how to install the TV: <https://youtu.be/zTGBbCbYONs>



### 3. Mini projector.

For multi-person large size cabin chamber, you can consider the mini projector for better experience during oxygen therapy. The projector size is small and no need to install it inside the chamber. You also can use it for your daily life.





#### 4. Massage chair.

Usually we will match with normal sofa inside the chamber, we can upgrade the normal sofa into massage chair according to your demand. The massage chair is more comfortable and it has different massage modes. You will feel the time flies so fast during oxygen therapy!



This is the video about how to assemble massage chair:

<https://youtu.be/NarKuSQfogg>

## 5. DC power supply system

If you buy any equipment we mentioned above, we will match with the DC power supply system freely. Certainly, if you want to buy it separately and connect with your DC electronic products, it is acceptable for us.



# The luxury hyperbaric chamber for high-end market:



Paramount P1



Paramount P4



Paramount P2



Paramount P5



Paramount P3



Paramount P7



This kind of hyperbaric chamber aims at high-end market, so **it is including the equipment like air conditioner/massage chair** and it allows inside user to lay down inside the chamber.

Comparing with the other models, this kind of luxury chamber have the advantages:

1. These luxury chambers have **sterilization function** which is popular for some public place like the clinic/hospital and so on. After using the chamber, you can close the door and press the sterilization button to start to sterilize inside chamber.
2. These luxury chambers use **automatic system**. When the door was closed, the concentrator would start to work automatically. This is what we called automatic system.
3. The massage chair we match with these luxury chambers is so comfortable and flexible. **The massage chair can become a bed to lie down**, so the chamber room become sleeping room. The massage chair also have different massage modes. You will fall in love with it when you experience the chair.





**DR.HUGO**

***Your Oxygen Therapy Expert***

POWER BY:



LANNX Bio&Med Co.,Ltd

Website: [www.lannx.net/hbot/](http://www.lannx.net/hbot/)

E-Mail: [sale@lannx.net](mailto:sale@lannx.net)

-Search our channel in YouTube to view operation video:

<https://www.youtube.com/@oxygensupplycenter>