

IMPORTANT

Instructions for Using

Warning:

- * The ports between the cabin and the oxygen concentrator needs to be connected correctly, it is prohibited to connect multiple tubes to the same port.
- * It is prohibited to use sharp objects to scratch or scrape the chamber.
- * It is prohibited to use hot objects to touch the cabin or to heat the chamber.
- * It is prohibited to use alcohol to wipe the outside of the cabin.
- * It is prohibited to change the zipper of the chamber without authorization.
- * It is prohibited to change the original oxygen supply system to other oxygen supply system without authorization.
- * It is prohibited to connect more oxygen supply system to the chamber without authorization.
- * It is prohibited to artificially close the automatic pressure relief valve.
- * It is prohibited to replace the emergency pressure relief without authorization.
- * It is prohibited to modify the oxygen supply system's control program without authorization.
- * It is prohibited to install any electric appliance without authorization. All the electric appliance we used like water-cooled air conditioner/TV/massage chair is DC(direct current) appliance.
- * It is prohibited to use AC(alternating current) appliance inside the chamber.
- * It is prohibited to stack other objects within 50 cm of the chamber.
- * keep away from bright and dark fire sources when operating system.
- * In order to maintain the life of the chamber, it is recommended that a single therapy session should not exceed 1.5 hours.
- * To ensure safety, it is recommended to do therapy in the company of other.

1. Please read the instruction manual and the instructions for use carefully before use.
2. Ear pressure relief methods
 - ① Pinch the nose and inflate vigorously.
 - ② Other methods: yawning (opening the mouth wide), swallowing movements, chewing gum, etc.
3. Exit method
 - ① **Normal**

Press the “OFF” button on the screen, the system will enter the shutdown procedure, and the status bar will display the shutdown status. After 30 seconds, the oxygen source host will stop, and the system pressure relief valve will automatically open. After 15-20 minutes, the system will automatically release the pressure to the minimum. You can open it then go out



of the cabin.

② Early exit

If you want to terminate the pressurization and leave the cabin in advance, please click the "OFF" button on the screen. At this time, the system will enter the depressurization stage. After the depressurization is completed, you will open the door and leave the cabin.

③ Quick pressure relief

When it is necessary to quickly release the pressure and get out of the cabin in an emergency, you can manually rotate the "emergency pressure relief" handle on the right side of the operation panel to speed up the pressure release speed, but it may cause discomfort such as ear pressure pain. Therefore, rapid pressure relief is not recommended for non-emergency reasons!

4. It is strictly forbidden to bring the following items into the cabin:

Matches, lighters, charging treasures and electronic equipment higher than 24V, other items that become fire sources, items that generate sparks when impacted, bottles/cans (soda water) with internal and external pressure differences, metal items, flammable items, foreign objects, Lethal weapons (knives, scissors, etc.), waterproof mobile phone electronic products, etc.

5. Do not use this product if you wear the following medical equipment

*** Implantable medical devices such as cardiac pacemakers and implantable defibrillators; wearable medical equipment such as electrocardiographs.**

6. Please consult a physician before using the oxygen chamber for the following persons:

Perioperative period of thoracic surgery, respiratory infectious virus infection, perioperative period of middle ear surgery, uncontrolled epilepsy, high fever, congenital spherocytosis, claustrophobia, skull base fracture with cerebrospinal fluid leakage, not recommended within 3 months of pregnancy Multiple hyperbaric oxygen therapy, uncontrolled hypertension, diabetic patients, angle-closure glaucoma), bullae, bradycardia (less than 50 beats/min), active untreated bleeding, tuberculosis cavitation, severe emphysema, Neonatal bronchopulmonary dysplasia.

7. The following persons must be accompanied by other equipment operators before using the oxygen chamber:

People with physical and mental disabilities (such as hearing and visual impairments), people who cannot understand the contents of the instruction manual, and children (must be accompanied by guardians when using).

8. Avoid using:

Untreated pneumothorax, taking disulfiram at the same time, taking antineoplastic drugs such as bleomycin, cisplatin, doxorubicin at the same time, premature and (or) newborns with low body weight, etc.

9. Repair and maintenance should be done by the original factory or its authorized professionals, and unauthorized disassembly and modification are prohibited.