



Instruction for Use

Product

Dressing Pack

Intended Use

It is used for clinical operation in medical institutions.

NO.	Name	Intended Use
1	Sterile Field	Used to package medical devices
2	Drape	Used to provide a physical barrier to protect the surgical area from contamination
3	Gauze Ball without X-ray	Gauze Ball without X-ray is used to cover/clean wounds or abrasions and absorb their exudates, absorb body-surface exudates. It is only intended to be used as a mechanical barrier; and not contact mucous membrane or manage the micro-environment of injured skin.
4	Non-woven gauze ball	Used for wound and wound cleaning and care
5	Cotton Ball	Used for cleaning skin and wounds.
6	Non-woven gauze pads	Used for surgical or local wound care by medical staff
7	Gauze Sponge without X-ray	Gauze Sponge without X-ray is used to cover/clean wounds or abrasions and absorb their exudates, absorb body-surface exudates. It is only intended to be used as a mechanical barrier; and not contact mucous membrane or manage the micro-environment of injured skin.
8	Hand Towel	Used for wiping hands
9	Absorbent Pad	used to absorb a large amount of exudate and keep the wound and skin surface dry. It is only intended to be used as a mechanical barrier; and not contact mucous membrane or manage the micro-environment of injured skin. It contains CELLULOSE
10	Purcotton swabs	used to cover/clean wounds or abrasions and absorb their exudates, absorb body-surface exudates. It is only intended to be used as a mechanical barrier; and not contact mucous membrane or manage the micro-environment of injured skin.
11	Medical Mask	Sterile Medical mask used for clinical medical personnel wearing during invasive procedures, and providing protection for receiving treatment patients and implementation of invasive operations medical personnel to stop the blood, body fluids and spatter spreading.

12	Disposable Examination Gloves	Used by medical staff to prevent cross infection between patients and users or to handle contaminated medical materials during the examination, diagnosis, and treatment process.
13	Plastic forceps	Used for gripping items during the diagnosis and treatment process
14	Disposable scissors	used to cut dressings
15	Plastic Trays	Used for storing, classifying, and transporting various medical equipment
16	PE apron	Used for effective isolation of pollutants, reducing the chances and risks of infection, and ensuring hygiene in the medical process
17	Plastic cup	Used for temporary storage of iodine or alcohol
18	Biohazard Bag	Used for storing and transporting medical specimens

Intended Patient Populations

Patients with exudate from wound, which including all age groups.

Age: no restrictions

Weight: no restrictions

Gender: no restrictions

Nationality: multiple

Intended user

Medical professionals in a specialty, clinical nurses

Intended using environment

This device is only designed for use in hospitals.

Potential complications

Infection;

Foreign body reaction

Medical Conditions

Wound care

Indications

Preparation for surgical operation

Contraindications

None

Cautions

- 1) Do not re-use;
- 2) Do not use if package is damaged;
- 3) Consult instructions for use or consult electronic instructions for use.

- 4) Keep away from sunlight, Keep dry
- 5) Please inform the manufacturer and competent authority in case of any adverse events related to the device occur.
- 6) This product is sterilized with EO and has a shelf life of 3 years. Please use it within the product's shelf life.

Model /Size

MG-DP-01, MG-DP-02, MG-DP-03, MG-DP-04, MG-DP-05, MG-DP-06, MG-DP-07, MG-DP-08, MG-DP-09, MG-DP-10

Storage conditions

Products should be stored in a sealed, moisture-proof, ventilated and dry place.

Disposal

The product is disposable and should be disposed of according to local medical waste regulations after use.

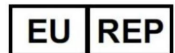
Shelf-life

3 years

Sterilization

The product is sterilized by EO. If the package is damaged, please do not use it

Definitions of symbol used

Signs	Description	Signs	Description	Signs	Description
	Trademark of Manufacturer		Batch number		Date of Manufacture
	Medical device		Model number		Country of manufacturer
	Use-by date		Sterilized using ethylene oxide		Caution
	Consult instructions for use		Catalogue number		Unique Device Identifier
	Single sterile barrier system		Double sterile barrier system		Keep away from sunlight
	Do not reuse		Do not use if the package is damaged		Keep dry
	CE marking of conformity, and Notified Body Code		Authorized representative in the European Community		Manufacturer

Manufacturer

Name: Haian Medigauze Co., Ltd.

Add: Jiaoxie Industry Park, 226633Haian, Jiangsu, PEOPLE'S REPUBLIC OF CHINA

Web: www.medigauze.com

Tel: 86+513-88240958

SRN: CN-MF-000009960

European Authorized Representative

Name: Shanghai International Holding Corp. GmbH (Europe)

Add: Eiffestrasse 80, 20537 Hamburg, Germany

E-mail: shholding@hotmail.com

SRN: DE-AR-000000001

Incident reporting

The user and/or patient shall report any serious incident that has occurred in relation to the device to Haian Medigauze Co., Ltd., and the competent authority of the EU Member State in which the user and/or patient is established.

Rev. A Issued date: 2024-04-20 (latest revision)