

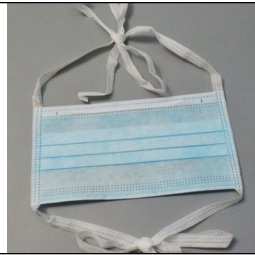


Instruction for Use

Product

Medical mask

Device Description and Specification

Device Name	Specification Length ×Width	Basic UDI-DI	Device figure
Medical mask	17.5cm×9.5cm, 17.5cm×9.5cm, 14.5cm×9.5cm, 14.5cm×9.0cm	697313440MGMM001L3	
Medical mask	17.5cm×9.5cm, 17.5cm×9.5cm, 14.5cm×9.5cm,	697313440MGMM002L5	

[Caution]

- 1) This product is for single use only, please do not reuse it.
- 2) Please check the integrity of the package before use, if the package is damaged, please do not use it.
- 3) Please read the instructions carefully before use. If the product has expired, please do not use it.
- 4) Please inform the manufacturer and competent authority in case of any adverse events related to the equipment failure.

[Use Method]

- 1) Wash hands
- 2) Take out the mask and face the white side of the mask to yourself, Put the side with the bridge of the nose upward
- 3) Put the ear bands on both sides of the mask over both ears

4) Adjust the bridge of nose to make it close to the bridge of nose.

[Storage conditions]

The product shall be stored in a dry, ventilated, non-corrosive environment and kept away from fire and flammable materials.

[Shelf-life]

3 years.

[Sterility]

EO sterilization

[Intended User]

Medical professionals in a specialty, clinical nurses

[Intended Patient Populations]

Patients who need to block external microorganisms from passing through respiratory tract infection, which including all age groups.

Age: no restrictions

Weight: no restrictions

Gender: no restrictions

Nationality: multiple

[Intended using environment]

Hospitals, health centers, clinics.

[Side effect]

Skin allergy, redness and swelling at the contact part, dyspnea

[Medical Conditions]

Need to block the spread of germs

[Indications]

Patients with exudate from wound.

[Contraindications]

It is forbidden for people who are allergic to non-woven fabrics

[Definitions of Signs]

Signs	Description	Signs	Description	Signs	Description
	Trademark of Manufacturer		Batch number		Date of Manufacture

Signs	Description	Signs	Description	Signs	Description
	Medical device		Model number		Country of manufacturer
	Use-by date		Sterilized using ethylene oxide		Caution
	Non-pyrogenic		Catalogue number		Unique Device Identifier
	Single sterile barrier system		Double sterile barrier system		Keep away from sunlight
	Do not reuse		Do not use if the package is damaged		Keep dry
	Do not resterilize		Authorized representative in the European Community		Manufacturer
	CE marking of conformity, and Notified Body Code		Consult instructions for use	/	/

Manufacturer

Name: Haian Medigauze Co.,Ltd.

Address: Jiaoxie Industry park,226633Haian, Jiangsu,PEOPLE'S REPUBLIC OF CHINA

Tel: +86-513-88240958

Web: www.medigauze.com

E-mail: produce@medigauze.com

SRN: CN-MF-000009960

European Authorized Representative

Name: Shanghai International Holding Corp. GmbH (Europe)

Add: Eiffestrasse 80, 20537 Hamburg, Germany



Document No.: MG/CE-MDR-05-02-02 A/0

E-mail: shholding@hotmail.com

SRN: DE-AR-000000001